


DIONavi. Membrane Lifter Precaution Instruction

Membrane Lifter

During DIONavi. Sinus case, cross infection may occur due to reuse of the Membrane Lifter.

The following precautions instruction is to prevent cross-infection.

CROSS-INFECTION

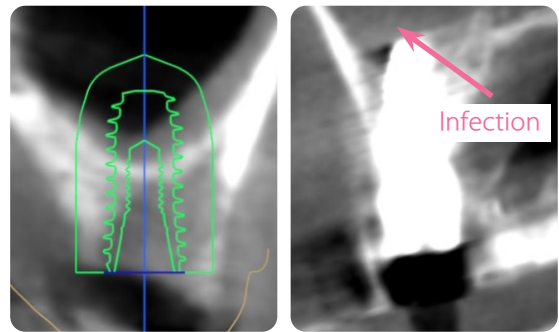

[PRECAUTIONS]

- » Membrane Lifter is for **single use only**, infection may occur.
- » Sterilize any medical device with an **autoclave** before using to a patient.
- » **B must be thrown away** after one use, and when A is reused, the inner surface **must be cleaned with an interdental toothbrush** immediately after surgery.

Membrane Lifter Can Be Purchased Separately

Product	Code	Image	Packing	Price(Set)	EA
Membrane Lifter	WML 6627			27,000	1
Membrane Lifter Tube	WML 6627-2			50,000	10