



Wax Denture

The product made a thermoplastic resin wax. It is easy to handle and get gums and occlusion of edentulous patients.



Maxilla



Mandible

- ✓ Patient does not need to visit twice.
- ✓ It can scan by oral scanner.
- ✓ Cheaper than making Splint.
- ✓ Reduce guide produce time.

How to use



1 Put during 10s in hot warter.



2 Align the inner surface to the patient's mouth.



3 Insert Impression material to the inner surface of Wax Denture.



4 Wax Denture put in mouth and check the bite.




5 Attach marker or flow resin on the buccal of Wax Denture.



6 Scanning opposite arch, Bite, Inner & Outer surface of Wax Denture.

Specification

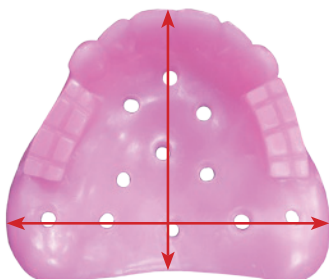

Wax Denture	Size		Code	
	Maxilla	Mandible	Maxilla	Mandible
Small	68.5 * 57.9mm	71.6 * 54.8mm	WDU 01	WDL 01
Medium	76.8 * 61.4mm	77.4 * 59.3mm	WDU 02	WDL 02
Large	79.5 * 62.5mm	80.2 * 62mm	WDU 03	WDL 03